



## Flexiguard 33 / Standard Heat Pad

This heat pad provides a very gentle heat and will work with the combined heat and body weight of your pet. In free air the heat pad may not feel very warm to the touch. To mimic the correct operating conditions place a large cushion of substantial weight (to represent your pet) on to the heat pad, leave the pad on for 30-60 mins and then feel the temperature on the surface of the cover. This will now be a comfortable temperature, close to body warmth.

### Warning:

- The Petnap must be used inside a Cosi Cover or with a soft covering on the top of the unit to spread the warmth, and a lining of material/newspaper/rug etc, underneath to retain the heat.
- Ensure that the heat pad does not cover more than 1/2 of the animal's available floor area – to allow the choice of sleeping on or off the pad.
- If your pet is old, ill or unable to shift position by themselves (including newborn puppies etc) please make sure you alter their position regularly, do NOT use with animals under anesthetic or similar medication.

### Instructions for use:

- Only use the heat pad for animals.
- We do not recommend the use of VETBED, cushions etc on top of the heat pad as this will limit the amount of heat passing through to the animal. Ideally the heat pads should be used with a light covering on top such as the Petnap cosi cover and some insulation under the pad. Use in a cool environment of 15c or under.
- The Petnap heat pad can be wiped down with a mild Milton solution.
- Do not immerse the heat pad in any liquids. Do not use with hay, straw, sawdust, etc.
- Ensure the Petnap heat pad is dry before use.
- Do not pull the Petnap heat pad by the power cord or use it as a handle.
- Carefully examine the Petnap heat pad before each use.
- Never continue to use your Petnap if it should become damaged. Return it to us and we will advise whether the unit can be repaired.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- Ensure the heating pad is completely dry before storage – store in a clean, dry place.
- We recommend always using in conjunction with an RCD and an external thermostat.
- The Flexiguard 33 only is suitable for animals which may chew it.
- The supply cord cannot be replaced. If the cord is damaged the appliance should be scrapped. Please note the base of the heat pad has the print on it & should be down.

#### Technical Specifications:

Input: AC 230/240V

Output: 8-10 Watts



1 Year Limited RTB Warranty After its useful life this item must be recycled and NOT placed in household waste